

INSTRUCTIONAL VIDEO



STEP 1: WHAT YOU NEED

1. Ur24T® TrueClr M unit
2. A Ur24T® recommended aspirator: A healthcare facility in wall aspirator, Drive Devilbiss Vacu-Aide® QSU Quiet Suction Unit Item # 7314P-D-UR24, Drive Devilbiss Compact Suction Pump.
3. Betadine swab (if desired) and gloves (if desired).

STEP 2: PREPARE

1. Wash hands with soap and water. Put on personal protective equipment (PPE) such as gloves (if desired).
2. Clean penis and wipe with betadine swab (including underneath foreskin, if applicable).
3. Adjust aspirator pump to lowest setting.
4. Attach distal TrueClr M tubing to negative pressure canister on aspirator pump.
5. Turn on aspirator.

STEP 3: APPLY

1. Invert (roll back) the edge of the TrueClr M rubber sleeve (Figure 1)
2. Grasp penis with one hand.
3. Using the other hand, apply the TrueClr M to tip of penis, unrolling the rubber sleeve to cover at least the first inch of penis (Figure 2).
4. If foreskin is present cover at least 1 1/2 inches (Figure 2).
5. Titrate up for optimum urine output and patient comfort.
6. Urine should begin to flow via TrueClr M tubing into collection canister within some time. (Figure 3)

STEP 4: REMOVE

1. When urine stops flowing, wait for 30 seconds.
2. To remove the TrueClr M, create a gap in the suction by inserting a finger between the TrueClr M and the penis. Carefully roll the TrueClr M off the penis (Figure 4)
3. Turn off aspirator pump.
4. Be sure to clean and dry the application area after use.
5. Remember to record urine amount, date, and time.
6. Remove the canister. Dispose of urine according to biosafety and infection control protocols and standards.
7. Clean the canister with soap and water.
8. Wash the TrueClr M with soap and water (or suggested Ur24Technology® anti-bacterial spray), then let dry.
9. Empty the bladder every 4-6 hours, or as often as your healthcare professional recommends.

TROUBLESHOOTING

Problem	Possible Cause(s) & Solutions
---------	-------------------------------

Inadequate Suction	• Canister lid not on tight	• Adjust suction level
	• Tubing not connected properly	• Tubing kinked or catheter clogged

Dispose of all components as infectious waste.

To avoid cross-contamination, follow infection control policies established by your healthcare facility.

Exposure to extreme heat or cold may affect the color of the TrueClr M but does not affect the functionality.

Do not use while patient is bathing. Do not drop into water or other liquid while in use.

All components are latex-free.

Indication for use: The Ur24Technology Urine Collection Apparatus is intended for non-invasive urine collection management in male patients.

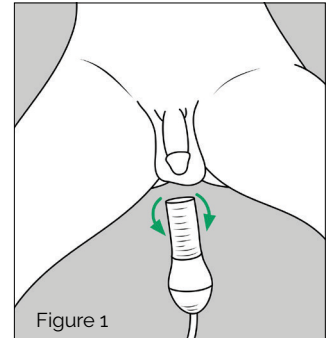


Figure 1

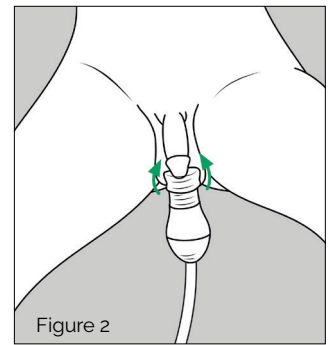


Figure 2

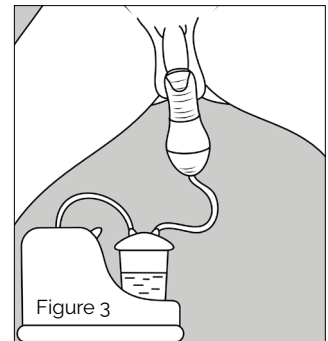


Figure 3

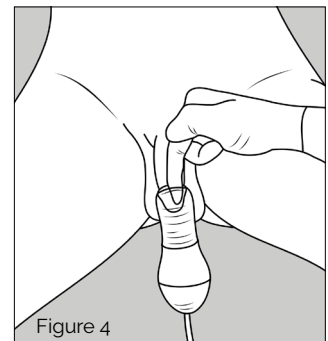


Figure 4



Scan Me

Ur24Technology.com

2022-06-HCP-M-Rev.A. • U.S. Patent Number 16/881601 and Number 15/416272 Class 1 Exempt Device, 21 C.F.R. § 876.5250

Questions or Feedback?

Please share your comments to make Ur24T® the best product for your patients
[Ur24Technology.com/feedback](https://www.ur24technology.com/feedback) or call 833-44-Ur24T

